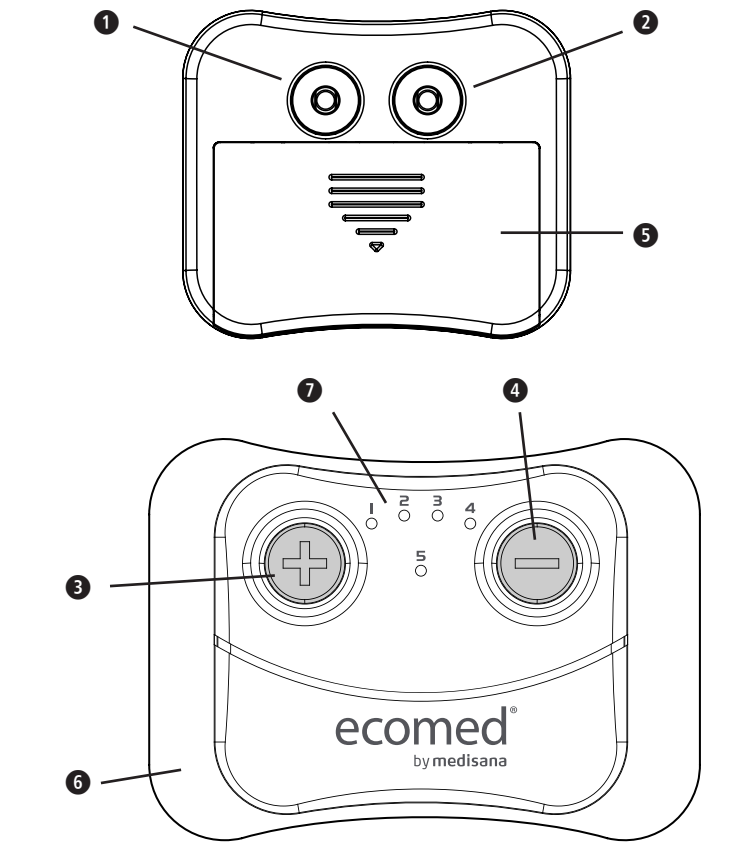
ecomed[®]
by medisana

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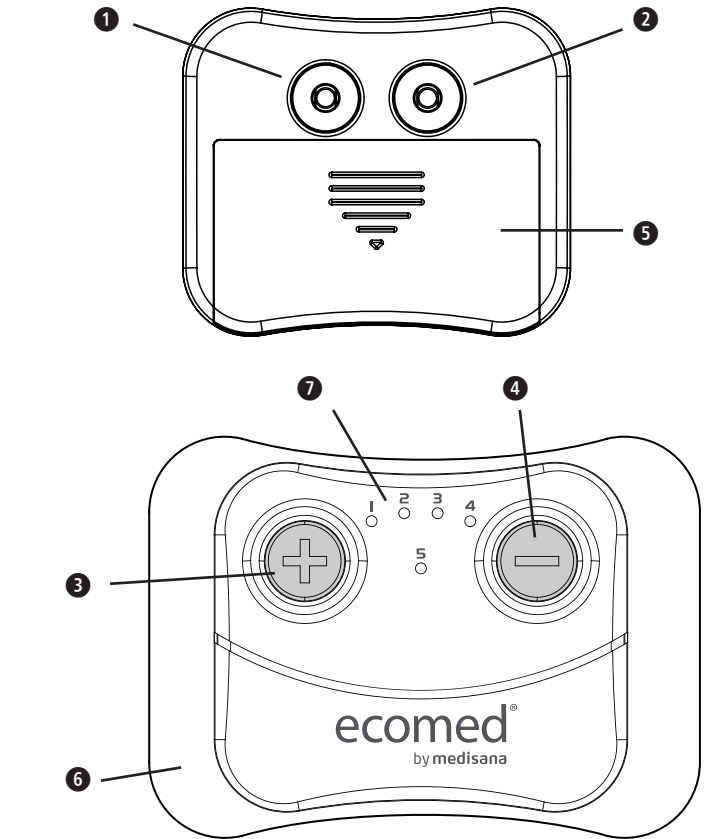
Device and controls

Explanation of symbols

This instruction manual belongs to this device. The instruction manual includes important information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the device.

WARNING
This symbol identifies hazards that may cause serious personal injury or death.

CAUTION
This symbol identifies hazards that may cause minor personal injury, product damage or property damage.

The device is protected against splashing water. Water splashed against the enclosure from any direction shall have no harmful effects. Disposal only according to the "WEEE" Waste Electrical and Electronic Equipment EC Directive.

Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.

Complies with the European Medical Device Regulation (EU) 2017/745.

Catalogue Number

Serial Number

Medical Device

Authorized representative in the European Community / European Union

Unique Device Identifier

Importer

Manufacturer

Electric shock protection type: Class II equipment

Batch code

Device classification: type BF applied part

Humidity range

Ambient pressure limitation

Temperature range

Date of manufacture

GB IMPORTANT INFORMATION! KEEP IN A SAFE PLACE!

Please read this instruction manual carefully, in particular the safety instructions, before using the device. Keep the instruction manual in a safe place for later reference. If you pass the device on to a third party, this instruction manual must remain with the device.

WARNING

- This therapeutic apparatus is not recommended to use in the following cases:
 - Do not use this device if the patient has any implantable electronic or metal equipment (e.g. heart pacemaker).
 - Epilepsy.
 - Severe lower extremity arterial circulation problems.
 - Abdominal or inguinal hernia site.
 - Do not use this device if you have a heart disease without consulting a doctor.
 - Do not use this device on pregnant women.
 - Do not use this device over, or in proximity to, cancerous lesions.
 - Having Fever.
 - Do not use therapeutic instruments in high humidity environments, such as bathing or showering in a bathroom bathtub.
 - Do not use after drinking.
 - Do not use low-frequency therapeutic instruments when using high frequency surgical instruments.
 - Don't use close to the genitals.
 - Recommendation: Do not use low-frequency therapeutic instruments near shortwave or microwave therapeutic equipment, where the output of the low frequency therapeutic instrument used is unstable. Keep a safe distance – at least 1 m – to such devices.
 - The high frequency surgical equipment and stimulator. When connected to a patient, burns may occur at the stimulator electrode and may damage the stimulator.
 - Abnormal feeling of the patient having medical treatment.
 - Patients with cardiac and neurological abnormalities.
 - Patients with body temperature exceeding 38°C.
 - Patients with infectious diseases.
 - Patients who can not express their will.
- Note:
 - Children must be supervised to ensure that they do not play with the device.
 - Make sure you are always aware of all warning items.
 - This product should not be used in the throat and mouth. This may result in severe throat muscle spasms and muscle contraction, which will likely block the airway and cause dyspnea.
 - This product should not be used in front of the chest, as the current introduced into the heart can lead to cardiac arrhythmia and cardiac arrest.
 - This product should not be used on the unattended patients, or those suffer from emotional distress, dementia, and with low IQ.
 - Use guidance has been listed in details; any improper use may lead to danger.
 - Do not use this device for any un-diagnosed pain symptoms. Do consult a doctor for diagnosis before use.
 - Electrodes can not be placed in areas that may cause current to pass through the brain (i.e. one-sided electrodes can not be placed on the front and back of the head at the same time, or relative to the left and right sides of the head).
 - Patients suffering from heart disease, cancer or other health-threatening illnesses should only use this device after consulting a doctor.
 - If the intensity of stimulation makes you feel uncomfortable, reduce the intensity of stimulation to a comfortable level. If the problem remains, please contact your doctor.
 - Do not use this equipment in the rooms where aerosol (spray) and pure oxygen are used.
 - Please do not use this equipment near any highly flammable substances, gases or explosives.
 - Please check the cable and related connections are in good condition before each use.
 - Turn off the device before using or removing the electrode pad.
 - This product can only be used with electrode wires and electrodes recommended by the manufacturer.
 - Use caution if stimulation is applied over the menstruating uterus.
 - Do not use the device for any other purpose except what it is intended for.
 - Do not use on any muscle or joint which has swelling or edema. It may worsen the pain or condition.
- **Consult your doctor before using the device, especially when:**
 - Acute disease, especially in patients suspected or suffering from hypertension, thrombosis or malignant mass.
 - Patients with various skin diseases.
 - Patients with chronic pain disorders (e.g. metabolic disorders) with reduced pain.
 - Patients taking medication.

- Adverse reactions
 - Skin allergies caused by electrode sheet pad gels and electrode irritation burns are potential adverse reactions. If allergic symptom appears on skin, please discontinue the treatment and consult your doctor.
 - If the intensity of stimulation makes you feel uncomfortable, reduce the intensity of stimulation to a comfortable level. If the problem persists, contact your doctor.
- Illustrations in this manual are effect drawings for reference only. Be sure to read this instruction manual before operation in order to use this device safely.
- Caution – Pay attention when using the electrode pad!
 - The electrode pad must not overlap with itself.
 - Do not bend the electrode pad.
 - Avoid touching the adhesive surface with your fingers.
 - After bathing, or after sweating due to exercises wipe the body parts to be treated before use.
 - Do not pull an electrode wire. This will damage the wires and the electrode pad.
 - Do not apply on damaged skin.
 - If the electrode pad is no longer sticky, please replace it.
 - The electrode pad must be used by one patient only.
 - If allergic reactions are experienced stop using the electrode pad immediately and consult your doctor.
 - Never tear off the electrode pad with force, e.g. if it got stuck to clothes. Try to apply some water to remove the electrode pad more easily.

BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Clean the battery and device contacts if necessary before putting in the batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Keep the battery compartment well sealed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Product structure and composition

- ①/② Fasteners for the electrode pad
- ③ **[+]** button (ON / increase intensity)
- ④ **[-]** button (OFF / Program select / decrease intensity)
- ⑤ Battery compartment
- ⑥ Electrode pad
- ⑦ LEDs indicating the active program (see also table ⑫)

Notes:

- If you notice any transport damage when unpacking, please contact your dealer immediately.
- First, check whether the device is complete and is not damaged. If in doubt, do not operate the device and contact your supplier or service centre.

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

WARNING
Please ensure that the polythene packing is kept away from the reach of children!
Risk of suffocation!

Intended use

The device is intended to temporary relief of pain, caused by sore and aching muscles after strain from exercises or household activities.
Intended users: Users must be 18 years or older and be able to control the device and understand the instructions. The device is intended for domestic use only, it is not intended for professional use in hospitals or other medical facilities.

Contraindications:

- Acute diseases, especially if the patient is suffering from hypertension, thrombosis or malignant mass.
- Patients with skin diseases.
- Patients with chronic pain disorders (e.g. metabolic disorders)
- Patients undergoing medication.

Working Principle

TENS, or transcutaneous electrical nerve stimulation, refers to the electrical stimulation of nerves through the skin. TENS is an effective non-pharmacological method for treating different types of pain from a variety of causes. The pain-relieving or pain-suppressing effect is achieved by inhibiting the transfer of pain to nerve fibres and by increasing the secretion of endorphins in the body, as their effect on the central nervous system reduces the sensation of pain. The method is scientifically substantiated and approved as a form of medical treatment. However, this method does not cure any physical problems, it only helps to control pain. It does not work for everyone, but for patients it is reducing or eliminating pain effectively.

Battery Replacement


Before starting any application, please make sure the battery power is sufficient and the setting is correct. When replacing the batteries, please make sure the **TT-10E** is switched off. Then open the battery cover and install two new batteries (see **Fig. ⑧**). Make sure that the polarity is correct!

Application






The **TT-10E** offers a choice between 5 programs. The program table ⑫ shows placement suggestions for the electrodes depending on the selected program. However, please note that each patient reacts differently to stimulation current therapies. Therefore, the positioning of the electrodes and the program selection for your requirements may differ from the therapies presented here. You should make a note of the optimum parameters for you (electrode position, program and current) for subsequent treatments.

The illustrations in the program table ⑫ show for each of the programs 1 to 5 examples of how to attach the electrodes to the treatment points that are possible with the respective program.

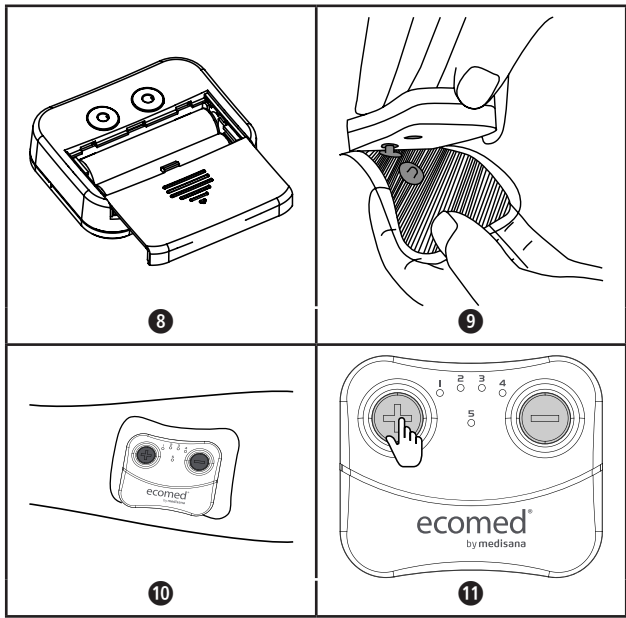Operation

- Install batteries (if necessary). – see Fig. ⑧
- Fasten the electrode pad onto the device. – see Fig. ⑨
- Paste the electrode pad (together with the device that has been fastened to the electrode pad) onto the part of the body that needs treatment. – see Fig. ⑩
- Press the **[+]** button to switch on the device and to select the intensity. – see Fig. ⑪
 - The first press on the **[+]** button switches the device on.
 - Then press the **[-]** button to select the required program (see also the program table ⑫). There are 5 LEDs ⑦ for the available 5 programs. The LED for the active program is lit. Program switching is done cyclically, i.e. program No. 5 is followed by program No. 1 again.
 - Each further key press on the **[+]** button increases the intensity, and each (short) key press on the **[-]** button decreases the intensity. There are 10 intensity levels available. If the device is already on the lowest intensity level and the **[-]** button is pressed again then the device enters the stand-by mode.
- The treatment ends automatically after approx. 30 minutes, then the device enters the stand-by mode.
- In the stand-by mode, the device is waiting for further buttons to be pressed. If no buttons are pressed within (approx.) 2 minutes the device is automatically switching off.
- Alternatively, you can stop the application immediately, i.e. before the regular 30 minutes have elapsed, by pressing the **[-]** button for approx. 2 seconds.

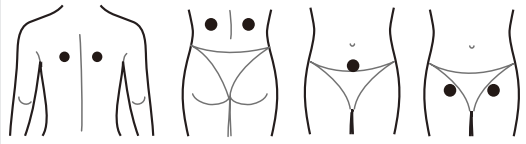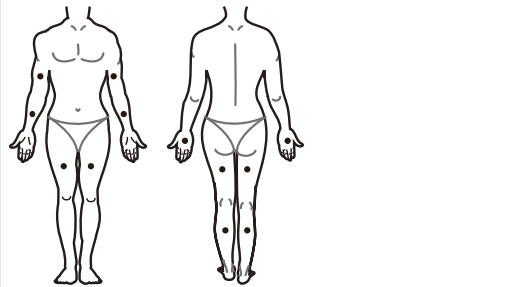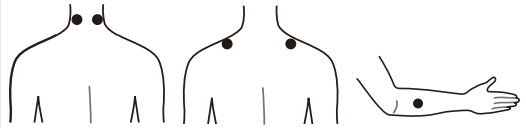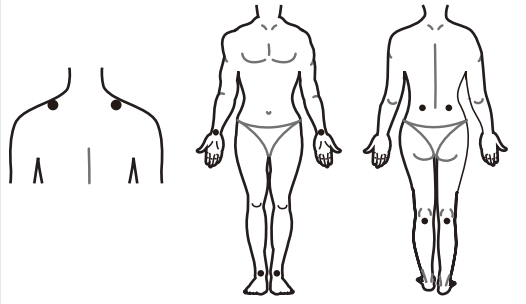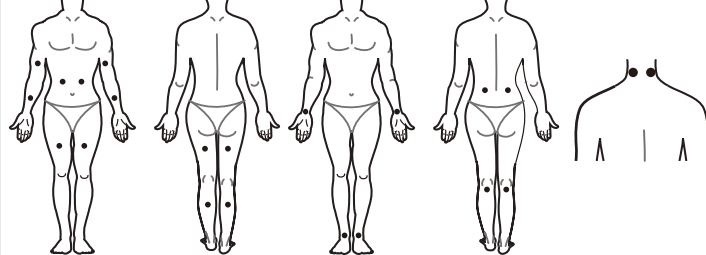⑩ Program table	
Prog.	Examples of electrode positioning
1	
2	
3	
4	
5	

After finishing the treatment

- Switch off the device and detach it from the electrode pad.
- Please carefully remove the electrode pad from the skin. Re-paste the protective film over the pad for protection. The electrode pad can become dirty very easily, which makes the viscosity decrease. Therefore, we strongly recommend to put the electrode pad to the original place after treatment.
 - Grab a corner and remove the electrode pad. Avoid to touch a large area of the adhesive surface.
 - Attach the electrode pad gently to the original protective sticker.
 - Never press the electrode pad with force, this will damage the surface.
 - Please spend care not to bend the electrode pad excessively.
 - Keep the electrode pad and the protective sticker clean.
- Store the device and the electrode pad in the original packaging. This prevents the parts from becoming dirty.

Maintenance and cleaning

- In order to ensure trouble-free and long-term functional reliability of the device, the following points should be observed:
- When the device is switched off, remove the electrode pad from the device and clean the device with a soft, slightly damp cloth. In the case of heavy soiling, you can also use a mild detergent.
 - Protect the device from moisture. Do not hold the device under running water and do not immerse it in water or other liquids.
 - If the adhesive surface of the electrode pad has become dirty so that it is no longer sticky you can try remove the dirt with a slightly damp soft cloth. This should restore the viscosity – at least temporarily. But note that too much moisture will permanently decrease the viscosity.
 - The electrode pad must not be immersed into water or be washed! Otherwise, the consequence might be a risk of electric shocks.
 - Do not place the device or the electrode pad on hot surfaces and do not expose them to direct sunlight. Do not keep them in places of extreme temperature, high humidity, or places with dusty or salty air, or locations with strong vibrations or mechanical shocks. Keep them away from places where chemicals are kept or corrosive gases are produced.
 - Keep the device and the electrode pad out of reach of children!
 - For hygienic reasons, each user should use their own electrode pads. We recommend to replace the electrode pad by a new one after approx. 50 applications.
 - Do not use aggressive substances or chemical cleaners to clean the device.
 - Make sure that no liquid gets into the device. Do not use the device again until it is completely dry.
 - Never clean the device during operation. Before cleaning, make sure that the device is switched off and the batteries have been removed!
 - You may only clean the device yourself. In case of malfunctions or damages do not repair it yourself as this leads to the automatic expiry of all guarantee rights. Ask your specialist dealer and only have repairs carried out by authorized service companies to avoid hazards.

Anomalies (Troubleshooting)

Issues	Possible causes	Solution
Unable to boot	1.The batteries are empty or incorrectly installed. 2. Malfunction	1. Check the batteries. Make sure that the battery type is correct. Make sure the batteries are correctly inserted. 2. Call for after-sales maintenance.
Automatic Stopping	1. The electrode pad has come off from the body. 2. Low batteries.	1. The electrode pad was not attached well. 2. Replace the batteries.
Weak electrical stimulation signal	Electrode issues: 1. Position problems. 2. The electrical contact between device and electrode pad is insufficient. 3. The electrical contact between electrode pad and the body is insufficient. 4. The skin is too dry. 5. Low batteries.	1., 2., 3. Replace / Reconnect 4. Use a damp soft cloth to moisten the skin where treatment is needed. 5. Replace the batteries.
Feeling unwell. The skin becomes red or tickles.	1. Too much intensity, or too long application time. 2. The electrical contact between electrode pad and the body is insufficient. 3. The electrode pad has become dirty or dry. 4. The skin is too sensitive toward the electrode pad.	1. Use only once per day, reduce the treatment time to a reasonable duration. 2. Replace / Reconnect 3. Try to wipe the electrode pad as described in section "Maintenance and cleaning". If this does not help, replace the electrode pad. 4. Check if this can be due to allergic reactions. Try to apply to some other part of the body. In case of strong reactions stop using the device and consult a doctor.

Technical Specifications

Name and model:	ecomed Wireless TENS Device TT-10E (model FDES105)
Output current:	max. 90 mA (at 500 Ω load)
Output voltage	max. 45VPP (at 500 Ω load)
Power supply:	3.0 V~, 2 batteries 1.5 V~ size AAA type Micro LR03
Load impedance:	500 Ω
Product service life	5 years
Operating environment:	Temperature: 41°F – 104°F / 5°C – 40°C, humidity: 30%RH – 85%RH, atmospheric pressure: 700 hPa – 1060 hPa
Storage environment:	Temperature: 14°F – 122°F / –10°C – 50°C, humidity: 15%RH – 90%RH, atmospheric pressure: 700 hPa – 1060 hPa
Type of waveform:	TENS / Bi-directional square-wave pulse
Output intensity:	0 – 10 levels (adjustable)
Output frequency:	2 – 80Hz
Output pulse width:	200 – 250µs
Treatment time:	30 minutes
Applicable part:	BF Application section. The electrode pad is type BF applied part.
Software version:	1.0
Protection level:	IP 22
Dimensions (L × W × H):	approx. 57 × 45 × 16.5 mm
Weight:	approx. 20 g without batteries / approx. 42 g with batteries
Size of electrode pads	small electrode pad: approx. 74 × 55 mm / large electrode pad: approx. 190 × 95 mm
Article number:	23455
EAN number:	4015588 23455 2

Replacement parts (available separately – not part of the TT-10E delivery):

- Set with 2 large and 2 small electrode pads, art. 23451 / EAN 4015588 23451 4

Disposal – Waste Electrical and Electronic Equipment (WEEE)

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the batteries before disposing of the device / unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- 1.The warranty period for **ecomed** products is 2 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- 2.Defects in material or workmanship will be removed free of charge within the warranty period.
- 3.Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- 4.The following is excluded under the warranty:
 - a.All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b.All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c.Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d.Applied parts which are subject to normal wear and tear.
- 5.Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The complete instructions for use are available for download at <https://docs.medisana.com/23455>.
Information about service can be found here: <https://www.medisana.com/servicepartners>

medisana GmbH,
Carl-Schurz-Str. 2, 41460 NEUSS, GERMANY

Famidoc Technology Co., Ltd.
Room 401, 501, 601, 901, Building 6, Zhongtang Tian'an Cyber Park,
No. 89, Jinyuan Road, Zhongtang Town, Dongguan City, Guangdong
Province, 523220, P.R. China
Website: www.famidoc.com

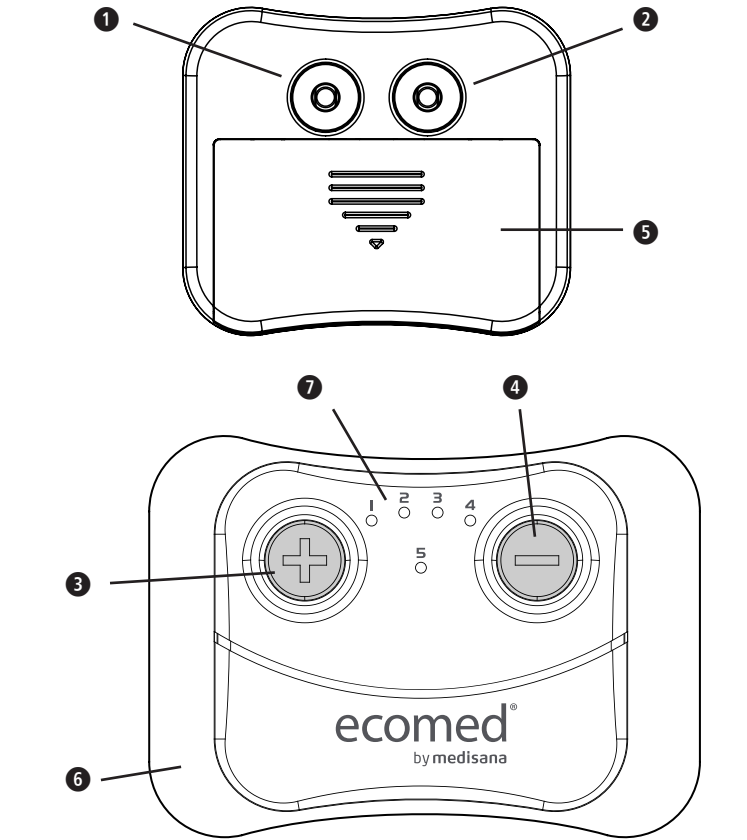
Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537 Hamburg, GERMANY

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